1

## Prepare for sample collection

Write the patient name, date of birth, phone number, and date of collection on the container(s), as well as on the label inside top pocket of the kit.

## Sample collection

Obtain scoops of stool from areas which appear bloody, slimy, or watery and enough to fill to the red line in the sample container(s). If the stool is formed (hard), take sample amounts from each end, and the middle. Mix with the contents in the container(s) with the spoon, twist the cap tightly closed, and shake until contents are well mixed.

2

3

## Package Specimen for Shipping

- Ensure sample container(s) are tightly closed and label is completed with patient information (see #1).
- Place container(s) in biohazard bag and then place bag with the containers into the bottom of the box.
- Activate instant cold compress and place it in the box.
- Please ensure that all paperwork is also included in the box before sealing and shipping.
- Place the box in the pre-labeled FedEx envelope.

Call FedEx at [1-800-463-3339](tel:1-800-463-3339) to schedule a same-day pickup. Please mention that you have a pre-paid FedEx express package.

Package may also be dropped off at any FedEx drop-off location.

Please note, FedEx does not pickup shipments on Saturday or Sunday.

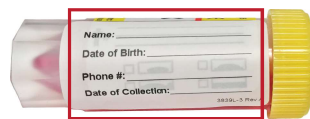
*We recommend collecting the sample and dropping off the shipment to a FedEx location on a weekday so your sample may be processed in a timely manner. Please retain a copy of your tracking information for reference.*

Follow the instructions on the back. Squeeze firmly and APPLY HIGH PRESSURE toward the inner fluid pouch to break.

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